

# retinEdge

AI FOR EYE CARE  
INITIATIVE FROM AMBERFLUX

# Sustainable Development Goal – Healthcare for all



UHC Index on essential service coverage is below 50%



3.5 Billion people have no access to basic and essential services



Diabetes expected to affect 1 in 10 people by 2045



India to become the world capital for Diabetes, followed by China



Impacting low income, and rural populations



SDG 2.2 End all forms of Malnutrition

# Current Diabetes Status



~540 Million have diabetes



Over 30% to 50% undiagnosed diabetics



Forecasted to reach a Billion in the next 20 years. 3 in 4 adults with diabetes in Low- and Middle-income countries.



Fastest spreading epidemic with ~2 Million deaths attributed to Diabetics.



35% of population do not believe in eye-care even if they have visual impairment



Less than 50% go for regular diabetic checkup



Diabetic retinopathy is the leading cause of new cases of legal blindness among adults



Over 10% of rural population is estimated to have diabetes and DR



India and China to have over 35% of global diabetics

# Why Retinopathy?



Non-intrusive detection



Eye is the only place where  
blood vessels can be  
photographed



Ongoing research for  
identification of life-threatening  
diseases like cardiovascular,  
diabetes, anaemia, etc.,  
besides identification of Eye  
related illnesses

# Introducing....

# retinEdge

AI FOR EYE CARE  
INITIATIVE FROM AMBERFLUX

retinEdge is a AI Based application for preliminary diagnostics of diabetes & other ailments using retinal images with a simple, low-cost, non-intrusive, intuitive approach

# Use Case: Detection



Susan lives in a remote village and wants her eyes checked without access to a nearby hospital or primary healthcare system. A local pharmacist/ Rural Medical Practitioner can use a fundus camera, take an image, and perform preliminary diagnostics.



Using retinEdge, the pharmacist/ rural medical practitioner can analyze the images for abnormalities and signals characteristics of diabetic retinopathy/ diabetes.



Analysis is done in real-time without human intervention, insights generated and relayed to specialist. Based on retinEdge Analysis, Susan can visit a specialist, if needed.



EdgeAI helps in reducing strain on health care system and network infrastructure as info is processed on device. EdgeAI also reduce the hospital waste for preliminary diagnosis.

retinEdge can also be used for regular, periodic checks for comparison from previous visit.

# Use Case : Notifications and Reminders



Best Care Inc is a regional hospital (RH) in the region and is tasked with Universal Healthcare goal for the region. It supports multiple Primary Health Care (PHC)/ Rural Health Centers (RHC) across the region. Still a large number of people in the region have no access to healthcare. Due to lack of diagnostic facilities, PHC/RHC are overloading the RH for many false positives.



Using retinEdge, PHC/RHC can perform preliminary diagnostics before referring patients to RH. Kiosks with retinEdge can be established at specific locations, and RH Doctors can converse with patients and guide them on usage of retinEdge and based on preliminary diagnostics can schedule appointments.



Using VoiceAI, RH can ensure the patients are provided timely advice, remind of their medicine timings, schedule appointments, and repeated visits. Also alert them of any medical camps. For remote patients, RH Doctors can provide telemedicine.



EdgeAI helps in ease of patient interactions and improves efficiency and increases revenue streams for hospitals and partners.

# Simplicity in Usage

Step 1:



Upload Retinal Image





Developed using 35000 Images from Eyepacs and Messidor database. In Discussions for Field Trials.

Currently running on AWS on cloud. Can run independently on a mobile/laptop/desktop without any cloud/internet connectivity, with images being uploaded at a later time to hospital database, enabling usage for medical camps, rural health centers, etc.,

Step 2: Select image file and click 'Analyze'

Upload Retinal Image

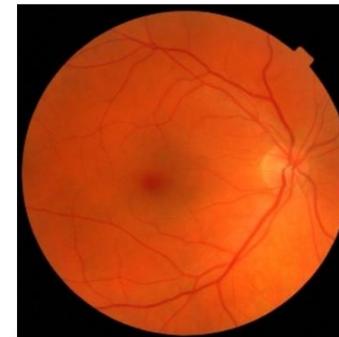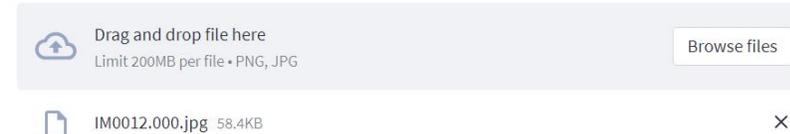


Analyse




Analyse

Step 3: Result

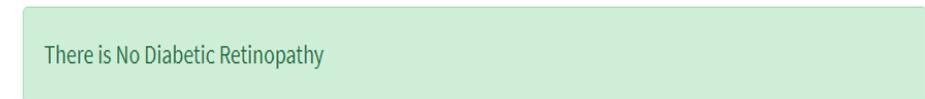


Disclaimer: This is only a preliminary diagnostic, based on the retinal images uploaded. Please consult your doctor for further diagnostics. Your doctor will recommend the right diagnostics and further steps. AmberFlux EdgeAI provides this as preliminary diagnostics and does not guarantee or warranty the accuracy of the diagnostics

# AmberFlux Solution Extension Flowchart

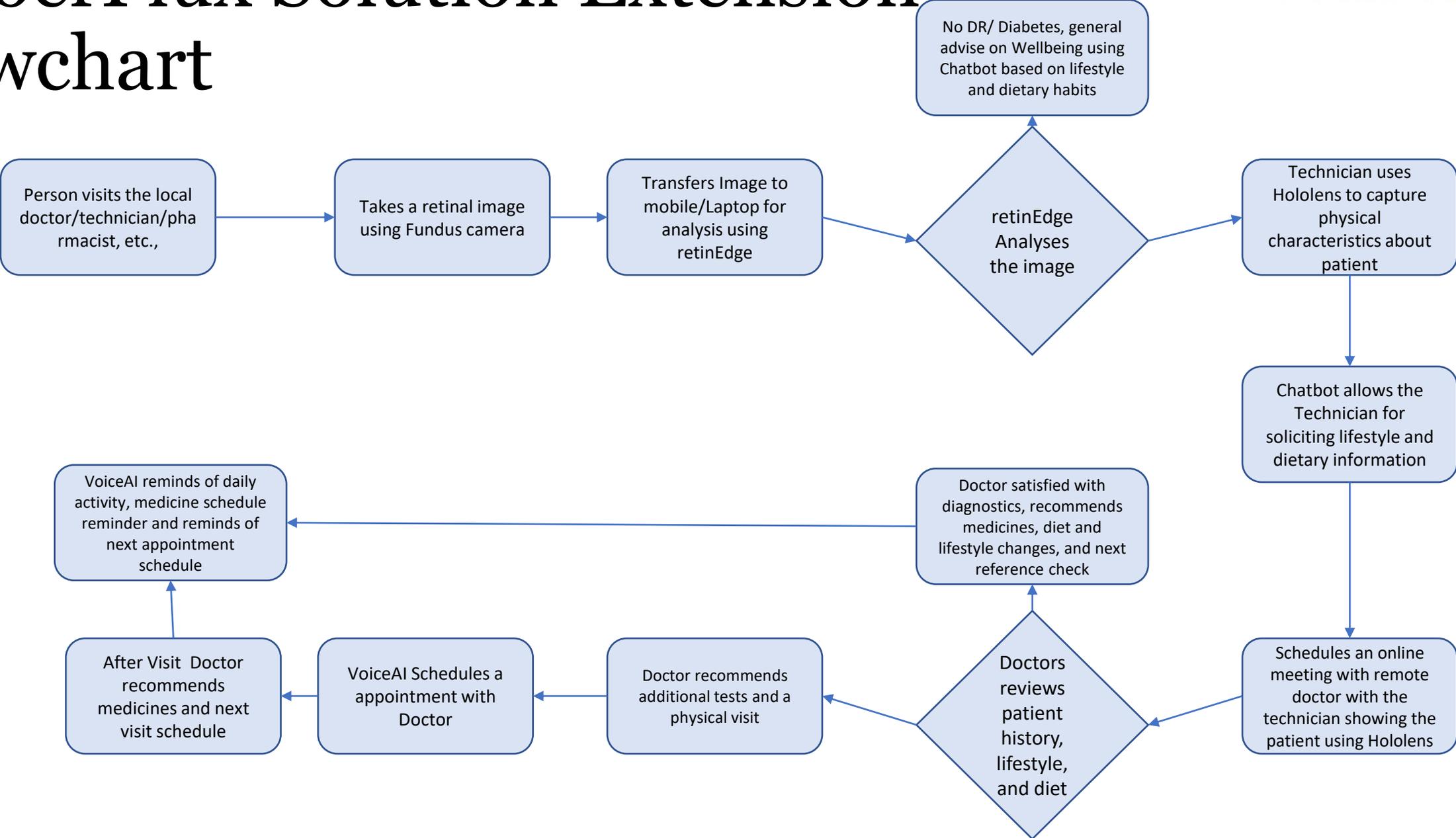

# AmberFlux Solution Extension & Continuous Improvement

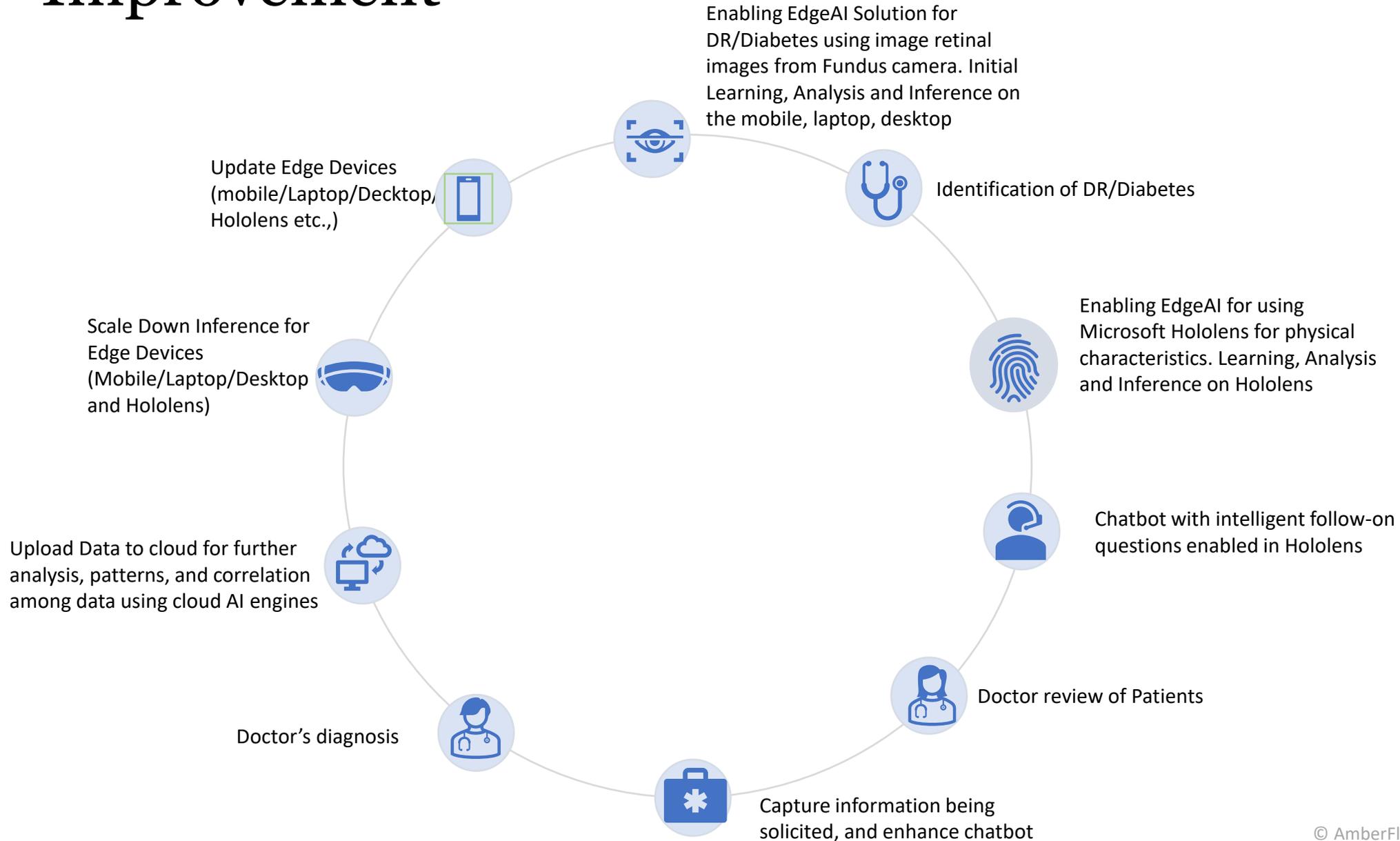

# Status



End-to-End  
Architecture &  
design ready



Diabetics  
Retinopathy  
product MVP is  
ready Analyzed  
35K Fundus  
Images for  
model  
development  
(from Messidor  
and EyePacs)




SaaS running on  
Cloud (AWS)  
Can run on  
mobile or  
desktop without  
connectivity



Discussions with  
Doctors for Field  
trials




Enhance product  
based on  
Doctor's  
feedback

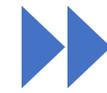

Enhance the  
model for better  
accuracy

# Business Model

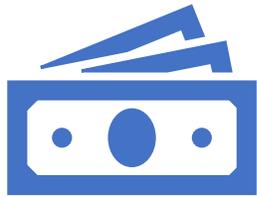

Pricing per eye analyzed  
Pay as you grow, P&L spent  
instead of expensive CapEx



Rapid sales to deployment  
cycles  
Customer can go live within  
days, rather than months



Brand building for sales and  
self service for delivery.  
Reduced cost of customer  
acquisition & improved  
profitability

# retinEdge

*Revolutionizing preliminary diagnostics using AI*

**Contact:**

Raghavendra Rao Gudipudi

[raghu@amberflux.com](mailto:raghu@amberflux.com)

(M) +91 9949039661